

---

## Enteralite Infinity Pump Green

---

Please refer to the Enteral Reference Guide: *Your Reference Guide to Home Tube Feeding* for further information.

To order formula and/or supplies call our Enteral Department at 888.648.2211 Ext. 6058.

### First Time Set-Up

- Turn pump on by pressing and holding the ON/OFF Button.
- Allow pump to go through power up sequence.
- Press + or - button to set Rate to \_\_\_\_\_ Dose to \_\_\_\_\_.
  - \* Remember to add at least 20ml more formula to first bag of the day, or each bag if bolus feed, for priming.
  - \* Clear volume between doses by pressing the volume button then the clear button.
  - \* If bolus feeds, rinse bag out and **"Prime"** tubing between feeds with water.

### Tube Feeding

- Pour formula into bag and hang bag on IV pole. The bottom of the bag should be at or just above the level of the door, or remove air and put in back pack.
- Place tubing into pump, close door.
- Turn pump on.
- Press and hold the **"Prime"** button until formula reaches the end of the tubing where the adapter is.
- Press volume button. Check to make sure it is "0". If not, clear volume on pump.
- Check tube placement with air if NG and/or flush the patient's \_\_\_\_\_ tube with \_\_\_\_\_ml of water. Flush as instructed by your physician or home care nurse.
- Connect the adapter to the patient's feeding tube. (Make sure it is secure).
- Verify that the correct rate is \_\_\_\_\_ & dose is \_\_\_\_\_.
- Press **"RUN"**.

### Key Features

- Pump will run nearly 24-hours once fully charged. There are four bars; each bar is six hours. It will beep and the pump will read **"Low Battery"** when there is one hour of charge remaining. The pump will then shut off automatically.

*Be sure to see the word **"RUN"** with a circle moving around it on the screen to confirm that the pump is running.*

**For 24-hour Pump Technical Support call 1.800.970.2337**