



Removing Air Bubbles from a Prefilled (Sodium Chloride & Heparin) Syringe




- 1** Inspect package integrity including protective wrapping, syringe and tip cap. Do not use if packaging is damaged, syringe is leaking or tip cap is not fully attached to syringe. Remove protective wrapping. Visually inspect the syringe and solution. Do not use if solution is cloudy or hazy.

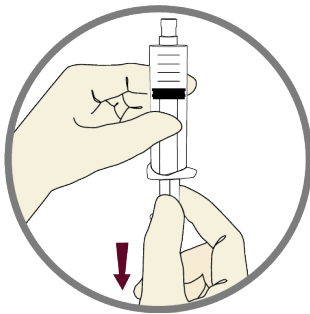

- 2** Before removing cap, hold syringe with tip cap upright. Pull plunger down; this motion breaks the seal between the plunger and the syringe.

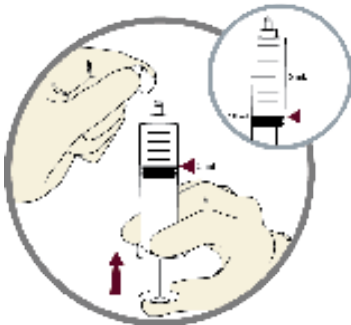

- 3** Remove tip cap with a twisting motion from the end of the syringe. Take care not to contaminate the syringe tip. Hold syringe with tip upward, tap side of syringe for bubbles to rise, slowly push syringe up until air and one or two drops of solution are expelled.

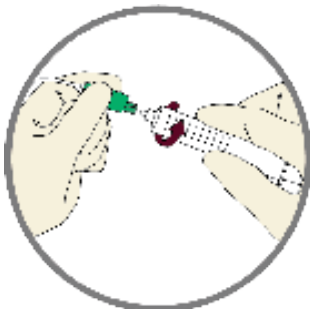

- 4** Flush site as directed.

For Single Use Only

Discard syringe and any unused portion of the solution in regular garbage.